PRODUCT NUMBER: 1032501
Honeywell SuperOne 3207 -
FFP3 NR D

FFP3 Non Reusable Moulded EN particulate respirator mask with a pre-shaped nose area and twin-stapled elastics. A "one size fits all" design. Boxes of 30 masks.

Honeywell

Overview

Reference Number

1032501

Product Type

Respiratory Protection

Range

Disposable APR

Line

Moulded masks

Brand

Honeywell

Brand formerly known as

SPERIAN

Industry

- Agriculture
- Automotive and Part Manufacturer
- Building and Construction
- Food Industries
- Foundry
- General Industry
- Iron and steel industry
- Laboratory
- Maintenance
- Medical and Pharmaceutical
- Metal steel
- Mining and Quarrying
- Water treatment
- Wood Industries

Product Use**Limitation to use:**

This single-use filtering half-mask should NOT be used in the following situations :

- oxygen concentration is less than 17% (oxygen-deficient atmospheres)
- contaminants or their concentrations are unknown or are immediately dangerous to life or health
- contaminants concentrations exceed levels fixed by applicable health and safety standards or 12 (10 in UK, FIN, D, I, S) x WEL*
- in the presence of toxic or radio-active contaminants
- in the presence of gases or vapours
- fire fighting.

Caution:

We recommend users to ensure that the level of protection offered by the respirator is sufficient for the type and concentration of contaminant(s) in the work area.

For use only by trained and qualified personnel.

Leak-tightness is unlikely to be achieved if the mask is worn against a beard or facial stubble.

This respirator does not supply oxygen.

* WEL : Workplace Exposure Limit.

Features & Benefits

Feature

Exclusive design and shape: semi-rigid mask, designed to stay maintained in time. Upper part pre-formed (no nose-bridge) for a quick and easy adjustment, highly efficient and secure Great inner volume to avoid any oppression on face and to ease communications Ultra-soft inner veil to offer a soft and pleasant contact to the skin and improve the breathing comfort Twin supple and resistant elastics, stapled on ear tabs to avoid any risk of irritation and of leakage in the filtering area Ultra-light(9g) to minimize wearer fatigue and increase acceptance in time No latex, no silicone, no PVC

Benefit

- Designed to fit everyone - Easy to use, comfortable and secure - Time saved due to minimum adjustment required - Breathing comfort is optimised - Compatibility with spectacles - Excellent field of vision - Prevents irritations - No risk of allergies to latex - No restrictive use - Environment-friendly

Technical Description

Standard

EN149.2001

Mask Type

Molded Cup

Protection Class

FFP3

Valve

No

Straps

Double elastics

Mask Size

Medium

Latex Free

Yes

PVC Free

Yes

Silicone Free

Yes

Shelf Life

3 years

Certifications

EU Category PPE

3

Quality Assurance

ISO 9001 / 2000

EU Certification

0082/236/079/03/19/0371

Certifications

- EN 149+A1:2009

REACH

| | |
|---|----------------------------|
| <p>EU Regulation 1907/2006 (REACH) Compliance</p> <p>The substance(s) listed below may be contained in this product above the threshold level of 0.1% by weight of the listed article.</p> | |
| Product Reference Sold in Europe | Substance > 0.1% |

| Substance Name | CAS Number |
|----------------------|------------|
| No Substance Content | |

Additional Certification Link

SuperOne series conformity attestation

Literature & Documents

How to fit a Sperian SuperOne mask

[http://www.honeywellsafety.com/supplementary/documents_and_downloads/respiratory_protection/single_use_\(disposable\)_respirators/26372/1033.aspx](http://www.honeywellsafety.com/supplementary/documents_and_downloads/respiratory_protection/single_use_(disposable)_respirators/26372/1033.aspx)

90025793-01_SuperOne_29L_EAC_HonNorth

http://www.honeywellsafety.com/supplementary/documents_and_downloads/secured/respiratory_protection/4295003426/1033.aspx


Photos & Images

S3203 HON 277x211

<http://www.honeywellsafety.com/assets/0/324/852/867/c6471173-a193-482a-8406-11de90ffa06e.jpg?n=6175>

>>> **SPERIAN SUPERONE RANGE - FITTING INSTRUCTIONS**

**HOW TO FIT
A SINGLE-USE MASK?**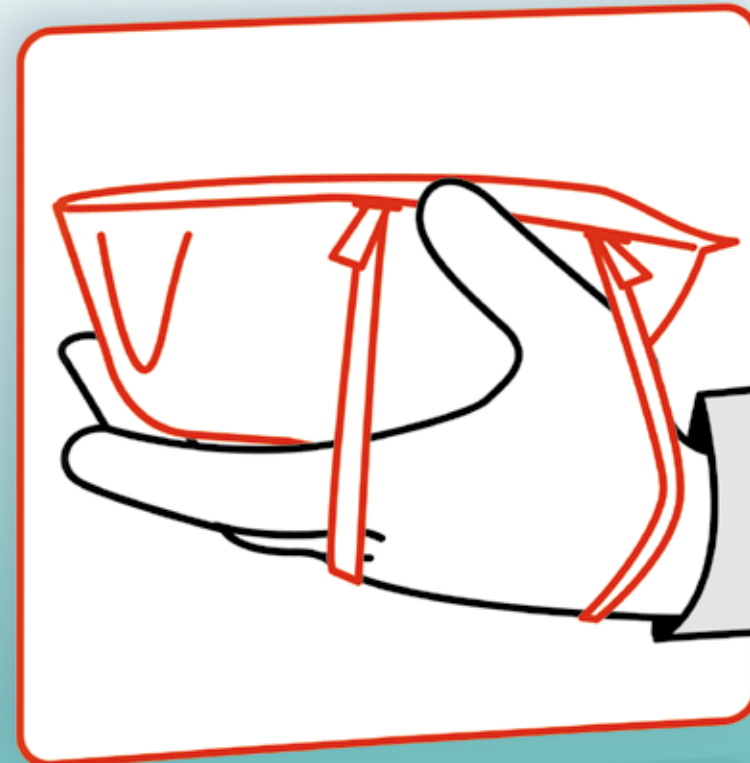

1

• Cup the mask in the hand.


2

• Put the mask on your face starting at your chin.
• First, place the lower headstrap around the neck below the ears, then place the upper headstrap above the ears.


3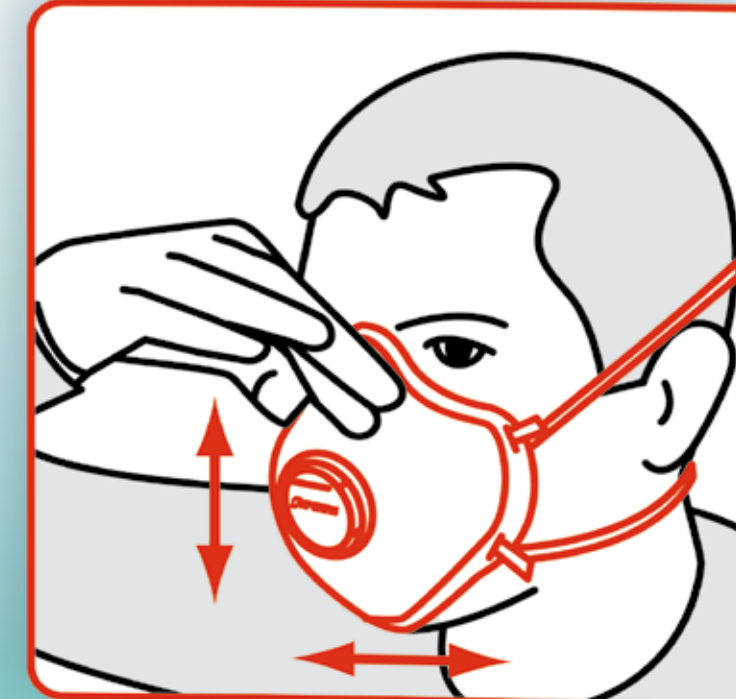

4

• Mould the nose area to the shape of the face.
• Move smoothly, right and left, up and down, in order to achieve the optimal position of the mask on the face.

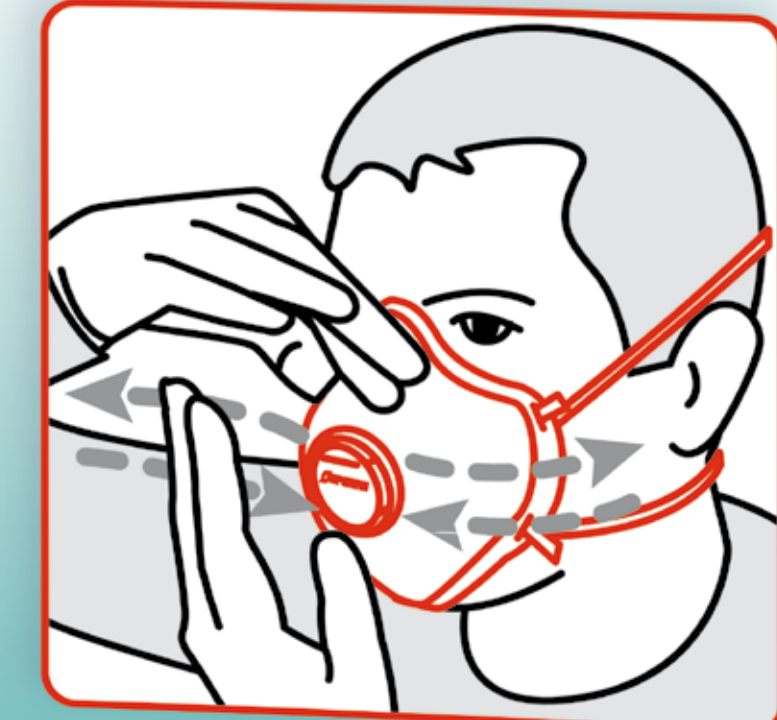5

• Check for leaks (Facial fit) as follows:
• Place both hands over the mask and inhale sharply.
• If you feel air entering around the edges, tighten the nose-piece and/or increase tension of the headstraps.

- + Valve membrane protected against impacts
- + Pre-formed upper part: excellent fit and leaktightness
- + No nose-bridge: very simple to use
- + Twin elastics stapled outside the filtering area
- + Latex, Silicone and PVC-free

+ Each mask meets the requirements of:
 > EN 149: 2001 standard
 > 120 mg loading test

■ ONE SIZE FITS ALL - AVAILABLE IN FFP1 AND FFP2 WITH AND WITHOUT VALVE

! WARNINGS:

- A beard may reduce the effectiveness of single-use masks
- The mask must be discarded after a single use (one shift maximum)
- A well fitted single-use mask guarantees efficient protection

SPERIAN PROTECTION EUROPE (SAS)
 Immeuble Edison - Paris Nord 2
 33, rue des Vanesses - BP 55288 Villepinte
 95958 Roissy CDG Cedex - France
 Phone: +33 (0)1 49 90 79 79
 Fax: +33 (0)1 49 90 71 38
 info-export@sperianprotection.com

SPERIAN PROTECTION (UK) LTD
 Unit 3 Elmwood
 Chineham Park Basingstoke
 RG 24 8WG - United Kingdom
 Phone: +44 (0) 1256 693 200
 Fax: +44 (0) 1256 693 300
 uksales@sperianprotection.com

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ENGLISH: INSTRUCTIONS FOR USE

APPLICATIONS: Filtering half-masks classified FFP1, FFP2 or FFP3 against solid and liquid aerosols in concentrations up to Protection Factor x WEL (Workplace Exposure Limit): 4 x WEL: FFP1 class (low toxicity), 12 [10 in UK, FIN, D, I, S] x WEL: FFP2 class (average toxicity), 50 [20 in FIN, S, UK & 30 D, I] x WEL: FFP3 class (high toxicity). « D » marked for passing the dolomite clogging test. « NR » (single-shift use) or « R » (Re-useable) marked according to the amended standard EN149:2001+A1:2009.

LIMITATIONS: Do NOT use this mask to enter or stay in an area where: • oxygen concentration is less than 17% (oxygen-deficient atmospheres) • contaminants and/or their concentrations are unknown or immediately dangerous to life or health • aerosol concentration exceeds levels fixed by applicable health and safety regulations or Protection Factor xWEL • gases and/or vapours are present. Do NOT use this mask in explosive atmospheres, except for references supplied with a Declaration of Conformity. Do not use before receiving training.

CHECKING BEFORE USE: The user shall ensure that the mask is suitable for the intended application before entering the contaminated area.

WEARING AND ADJUSTMENT: 1 – Cup the mask in the hand, the headbands downward. 2 – Place the mask under the chin. Place the lower headband around the neck, below the ears. 3 – Holding the mask against the face with one hand, place the top headband above the ears, around the crown of the head. 4 – CHECK FOR LEAK-TIGHTNESS AS FOLLOWS: • place both hands over the mask and – exhale sharply for a mask without exhalation valve – inhale deeply for a mask with exhalation valve • if you feel air leaks around the edges, readjust the mask by increasing tension of the headbands (by tying a knot) • repeat the procedure until satisfactory face-fit has been achieved. N.B.: requirements

for leak-tightness are unlikely to be achieved if the respirator is worn against a beard or facial stubble.

USING: Taking a new mask from its box. 1 – Check the condition of the mask before use. 2 – Fit the mask and check leak-tightness before entering the contaminated area. 3 – Wear it for the whole duration of exposure to contaminants. 4 – Discard it and replace with a new one if: • it is removed while in a contaminated area • excessive clogging causes breathing difficulty or discomfort • it becomes damaged • for masks offering relief against nuisance odours, the smell becomes detectable. Do not alter the mask. Leave the contaminated area if dizziness, irritation or other distress occurs.

WARNING: The mask shall not be used for more than one shift if it is for single-shift use (« NR »). Keep unused masks in their closed box and store in a non-contaminated area.

CLEANING: Applicable only on the re-useable masks (« R »). If the mask is to be used for more than one shift, the face seal must be cleaned with a wipe impregnated with a cleaning and disinfecting solution, without alcohol. Products comply with the European Parliament and Council Regulation (EU) 2016/425 for Personal Protective Equipment and subsequent amendments. EU-type examination certificate and Quality assessment following module D by APAVE SUDÉUROPE SAS - CS60193 - 13322 MARSEILLE Cedex 16 - FRANCE. Notified Body No. 0082. EU-type examination certificate issued to: "HONEYWELL RESPIRATORY SAFETY PRODUCTS". The EU declaration of conformity is available on the website <https://doc.honeywellsafety.com/>.

Sort out your waste by respecting the regulations in force.

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